




# Verpackungshinweise für Standard-Sterilisationskartons / Packaging instructions for standard sterilization cartons

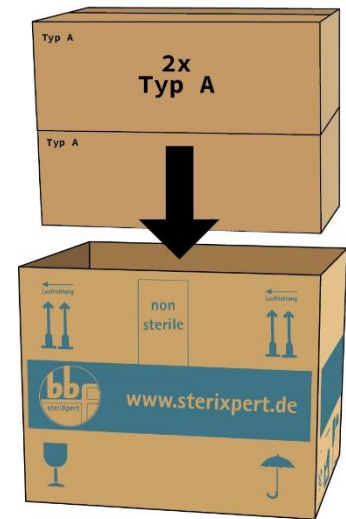

Der B-Karton (Art.-Nr.:200004) passt viermal in den Standard-Sterilisations-karton.  
Der A-Karton (Art.-Nr.: 200003) passt zweimal in den Standard-Sterilisations-karton. /  
*The type B carton (Item no. 200004) fits four times in the standard sterilization carton. The type A carton (Item no. 200003) fits twice in the standard sterilization carton.*



Der Standard-Sterilisationskarton (Art. 200001) muss vollständig mit festem Material z.B. Styroporplatten gefüllt sein. /  
Teilweise befüllte Kartons können ansonsten nicht bearbeitet werden.  
*The standard sterilization carton (Item no. 200001) must be completely filled with solid material, e.g. Styrofoam plates. Partly filled cartons can otherwise not be processed.*

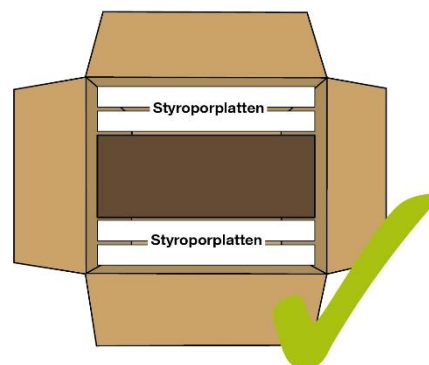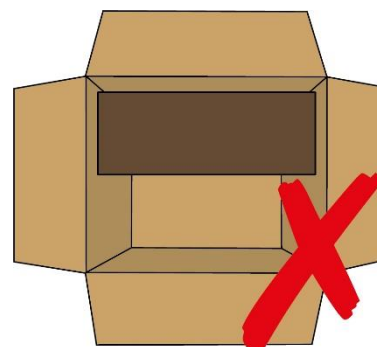

**Verpackungshinweise für Standard-  
Sterilisationskartons /**  
*Packaging instructions for standard sterilization  
cartons*



Die Standard-Sterilisationskartons sind mit Klebestreifen aus Kraftpapier (Art.-Nr.: 200010) zu verschließen. Die Länge der Klebestreifen zum Verschließen des Deckels und Bodens sollten etwa 75 cm betragen und ca. 10 cm überlappend and den Stirnseiten angebracht werden. Die ca. 35 cm großen Klebestreifen sollten anschließend mittig angebracht werden. /

*The standard sterilization cartons must be sealed with adhesive tape made of kraft paper (Item no. 200010). The length of the adhesive tape to close the lid and bottom should be around 75 cm and be placed with 10 cm overlap on the front sides. The approx. 35 cm adhesive strips should be arranged centrally.*



**Kunststoffklebestreifen sind nicht geeignet und führen zu Anlagenstörungen!! / Plastic adhesive strips are not suitable and lead to system failures !!**